HEINE QUALITY
MADE IN GERMANY

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HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0
Fax +49(0)81 52/38-202
E-Mail: info@heine.com · www.heine.com
MED 113158 2017-05-02

V-200.00.139

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DEUTSCH	3
ENGLISH	6
FRANÇAIS	9
ESPAÑOL	12
ITALIANO	15
SVENSKA	18
NEDERLANDS	21
DANSK	24
NORSK	27
SUOMI	30
PORTUGUÊS	33

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Please read and follow these instructions for use and keep them for future reference.

Intended Use

The HEINE® mPack mini rechargeable battery is intended for use with HEINE medical devices exclusively.

It may be used by professional medical staff and in professional healthcare institutions only.

For U.S. only:
Federal law restricts this device to sale by or on the order of a Physician or Practitioner!

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Product overview

- 1 Micro-USB connector
- 2 Socket
- 3 Controller
- 4 Status indicator

Setting up

Charge the HEINE® mPack mini fully before its first use.

Charging

Use a USB cable to connect the USB connector (1) of the mPack mini to a USB power supply (e.g. HEINE E4-USB, PC or Laptop). At the start of the charging process, all three illuminating segments of the status indicator (4) light up briefly. During the charging process, the segments light up progressively to indicate the charging status. As soon as all segments are on permanently, the mPack mini is fully charged. The indicator is switched off after a short period of time.

Installation

Plug the supply cable of the HEINE instrument you wish to operate with the mPack mini into socket (2).

Operation

Rotate the controller (3) to change the brightness of the connected instrument. To switch the mPack mini off, rotate the controller counterclockwise until you hear a „click“.

When the mPack mini is switched on and an instrument is connected to it, the status indicator indicates the charging status as follows:

- All segments are on permanently: fully charged
- Lower two segments are on permanently: sufficiently charged
- Lower segment (orange) is on permanently: almost discharged
- Lower segment (orange) flashes: strongly discharged, please recharge!

If no instrument is connected to the mPack mini when the mPack mini is switched on, the status indicator lights up for approximately 1 second and is then switched off.

The connected instrument cannot be operated during the charging process! The illumination of the instrument is switched off as soon as the mPack mini is connected to a USB power source. Once the charging connection is disconnected, the instrument illumination is switched on again at the previously set brightness level.

If the connected instrument is defective, the mPack mini shuts down automatically and issues an error-indicating sequence of flashes.

Use only CE marked USB power supplies (5 V) from reputable manufacturers that conform with the Class II and safety requirements from the IEC 60950-1 Information technology or IEC 60601-1 Medical electrical equipment.

Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: non-critical

Spaulding Classification USA: non-critical

Before cleaning, disconnect the device from the power supply.

In the event of suspected contamination, carry out hygienic reprocessing of the instrument.

The described reprocessing measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the reprocessing agents and procedures named in these instructions for use.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.

Do not use any spray disinfectant, or any dripping-wet or heavily-foaming cloths.

Do not reprocess automatically.

mPack mini

Cleaning and disinfection

(Wipe clean and wipe disinfect)

In the presence of contamination or for risky applications, take the mPack mini from its belt clip and reprocess the belt clip separately (see below).

Clean and disinfect the mPack mini by wiping it.

The cleaning and disinfection must be done with special care at exposed sites such as, e.g., the operating elements.

During the reprocessing, hold the mPack mini appropriately such that its openings face downwards.

Recommended agents

Cleaning agent: enzymatic (e.g. neodisher® MediClean)

Disinfectant agent: alcoholic (e.g. Incides® N), quaternary ammonium compounds (e.g. Mikrobac® tissues), agent ortho-phthalaldehyde (e.g. Cidex® OPA) or hydrogen peroxide (e.g. PREempt® Wipes)

Belt Clip

Cleaning and disinfection

Wash the belt clip (without the mPack mini) with warm water and a cleaning agent.

Dry the clip and then disinfect it by immersion disinfection.

Recommended combinations of reprocessing agents

Cleaning agent: enzymatic (e.g. neodisher® MediClean) and

Disinfectant agent: quaternary ammonium compounds (e.g. neodisher® Septo MED)

Maintenance

The device is maintenance free.

We as manufacturer do not stipulate a safety inspection in accordance with § 6 Medical Device Operator Ordinance (MPBetreibV), ref. attachment 1, for the HEINE product mPack mini.

The expected operational life of the HEINE® mPack mini is 7 years.

Battery replacement

A change of the charging cell is only necessary if it does not take the required charge anymore. With proper care this is usually after around 2 to 3 years. If a replacement is required, please contact your authorized dealer.

Service

The device does not require service.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments)

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

The device must not be placed near strong Magnetic fields, e.g. MRI units.

If the power source is transferred from a cold to a warm environment, let it rest for approximately 1 h to allow it to equilibrate to the ambient conditions.

The device should never be used or stored outdoors, since this may lead to malfunction.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

Do not kink the cable and do not wrap it around the power supply since this reduces the service life of the cable.

Disposal

The product must be recycled as separated electrical and electronic devices.

Please observe the relevant state-specific disposal regulations.

Dispose at your local collection point.

Electromagnetic Compatibility

Medical electrical devices are subject to special precautionary measures with regards to electromagnetic compatibility (EMC). They are to be installed and operated in accordance with EMC guidelines contained in the accompanying documentation of the medical device.

The appendix contains following tables

- Technical specification
- Explanation of the used symbols

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Technical specification

Environmental conditions for operation	+10 °C to +35 °C 10 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 10 % to 90 % rel. humidity 500 hPa to 1060 hPa
Recommended conditions for storage for more than 3 months	+5 °C to +20 °C up to 70 % rel. humidity
Input	5 V, 500 mA (Micro-B USB)
Output	max. 450 mA, max. 2 W
Nominal voltage	0 V to 5.5 V
Charging time	typ. 4 h, max. 10 h
Weight	95 g
Dimensions	44 mm x 23 mm x 101 mm
Class	Charging: II operation: internally powered
Housing protection	IP40

THIS INSTRUMENT DOES NOT WITHSTAND THE TEMPERATURE CONDITIONS FOR STORAGE ACCORDING TO ISO 15004-1. DO NOT STORE THIS INSTRUMENT AT CONDITIONS UNDER WHICH THE TEMPERATURE MAY BE ABOVE 45 °C OR BELOW 5 °C.

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Explanation of utilized symbols

The following symbols are used on the device or on the packaging:

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